

DARTMOUTH DENTAL PRACTICE

24 Victoria Road
Dartmouth
Devon
TQ6 9SA
01803 835 418

Marian Roberts BDS (Sheffield) GDC 61688
Aleksander Srokosz BDS (Plymouth) GDC 258317
admin@dartmouthdental.co.uk

www.dartmouthdental.co.uk

Change Log

Thursday, 1 August 2019

Bottle should be left inverted to dry

Saturday, 27 June 2020

Reviewed - some cosmetic changes
changed file name to Disinfecting Lines.
Change log amended.

MR Surgery - Disinfecting Lines

- Switch off air supply i.e. water bottle switch on right is turned to 'OFF' and water supply switch on left is turned to 'CITY'.
- Disconnect water bottle and dispose of contents.
- Using Unodent Dental Water Supply Cleaner Concentrate, as per manufacturers instructions. Measure 20ml of concentrate in measuring lid, and add to 1 litre of lukewarm water. Add this to water bottle and reconnect to the dental unit and restore the air supply ie. turn right switch (water bottle) to ON and left switch (water supply) to 'BOTTLE'.
- Close the door, and run the lines through for 2 minutes by pressing and holding button down on bottom left of panel.
- Leave solution in lines for minimum 20 minutes (can be left over night or over the weekend)
- Turn off air supply through bottle i.e. right switch to 'OFF' and left switch to 'CITY'.
- Remove bottle and discard remaining solution.
- Rinse the bottle thoroughly with water and leave to dry.
- After minimum of 20 minutes run lines through again with water for at least 45 seconds.
- Remove bottle and leave inverted to dry
-
- Switch off air supply and remove the bottle for the ultrasonic scaler.
- Disconnect water bottle and dispose of contents.
- Using Unodent Dental Water Supply Cleaner Concentrate, as per manufacturers instructions. Measure 20ml of concentrate in measuring lid, and add to 1 litre of lukewarm water. Add this to water bottle and reconnect to the dental unit and restore the air supply.
- Purge the line to expel any residual air until the solution appears.
- Leave solution in lines for minimum 20 minutes (can be left over night or over the weekend)
- Turn off air supply for the bottle.
- Remove bottle and discard remaining solution.
- Rinse the bottle thoroughly with water, and refill with clean water.
- Reconnect the bottle and air supply and purge the line for at least 45 seconds.
- Disconnect the air supply and remove bottle, rinse with clean water and leave bottle inverted to dry.

DARTMOUTH DENTAL PRACTICE

24 Victoria Road
Dartmouth
Devon
TQ6 9SA
01803 835 418

Marian Roberts BDS (Sheffield) GDC 61688
Aleksander Srokosz BDS (Plymouth) GDC 258317
admin@dartmouthdental.co.uk

www.dartmouthdental.co.uk

AS Surgery - Disinfecting Lines

- Switch off air supply i.e. water bottle switch on right is turned to 'OFF' and water supply switch on left is turned to 'CITY'.
 - Disconnect water bottle and dispose of contents.
 - Using Unodent Dental Water Supply Cleaner Concentrate, as per manufacturers instructions. Measure 20ml of concentrate in measuring lid, and add to 1 litre of lukewarm water. Add this to water bottle and reconnect to the dental unit and restore the air supply ie. turn right switch (water bottle) to ON and left switch (water supply) to 'BOTTLE'.
 - Close the door, and run the lines through for 2 minutes by pressing and holding button down on bottom left of panel.
 - Leave solution in lines for minimum 20 minutes (can be left over night or over the weekend)
 - Turn off air supply through bottle i.e. right switch to 'OFF' and left switch to 'CITY'.
 - Remove bottle and discard remaining solution.
 - Rinse the bottle thoroughly with water and leave to dry.
 - After minimum of 20 minutes run lines through again with water for at least 45 seconds.
-
- Switch off air supply and remove the bottle for the ultrasonic scaler.
 - Disconnect water bottle and dispose of contents.
 - Using Unodent Dental Water Supply Cleaner Concentrate, as per manufacturers instructions. Measure 20ml of concentrate in measuring lid, and add to 1 litre of lukewarm water. Add this to water bottle and reconnect to the dental unit and restore the air supply.
 - Purge the line to expel any residual air until the solution appears.
 - Leave solution in lines for minimum 20 minutes (can be left over night or over the weekend)
 - Turn off air supply for the bottle.
 - Remove bottle and discard remaining solution.
 - Rinse the bottle thoroughly with water, and refill with clean water.
 - Reconnect the bottle and air supply and purge the line for at least 45 seconds.
 - Disconnect the air supply and remove bottle, rinse with clean water and leave bottle to dry.

LP Surgery - Disinfecting Lines

- Switch off air supply and remove the bottle for the ultrasonic scaler and 3 in 1.
- Disconnect water bottle and dispose of contents.
- Using Unodent Dental Water Supply Cleaner Concentrate, as per manufacturers instructions. Measure 20ml of concentrate in measuring lid, and add to 1 litre of lukewarm water. Add this to water bottle and reconnect to the dental unit and restore the air supply.
- Purge the line to expel any residual air until the solution appears.
- Leave solution in lines for minimum 20 minutes (can be left over night or over the weekend)
- Turn off air supply for bottle
- Remove bottle and discard remaining solution.
- Rinse the bottle thoroughly with water, and refill with clean water.
- Reconnect the bottle and air supply and purge the line for at least 45 seconds.
- Disconnect the air supply and remove bottle, rinse with clean water and leave bottle to dry.
-